Troubleshooting List

This Troubleshooting List includes brief references to the comprehensive manual that can be downloaded from our website <http://www.birthalarm.com>. These references take the following format: “(see 2.3)”

The Birth Alarm Mobile Premium/Lite does nothing at all.

- Empty/low batteries?
 - *Install new batteries every season. Always replace all 3 batteries at the same time. If, for example, you fit 2 new batteries and leave 1 old one in place, this considerably hinders the operation of the device. (see 5.1)*
- Batteries installed correctly?
 - *The batteries should be installed as illustrated on the base of the battery holder. The “-” pole is furthermore always placed on the spring. (see 5.1)*
- Has the cover been correctly fitted on to the birth alarm?
 - *The cover of the Birth Alarm Mobile Premium/Lite only fits onto the Birth Alarm in one way. The light guides in the cover should be situated above the LEDs so that the coloured light of the GSM and status LEDs visible. (see 5.2)*

The Birth Alarm Mobile Premium/Lite fails to establish a network connection: (i.e. the GSM LED continues to blink rapidly and the STATUS LED only flashes blue).

- Has the SIM card been set up so that a PIN is required at start-up?
 - *When you insert the SIM into your own telephone and switch your handset on, the telephone must not ask for a PIN when switched on. If it does ask for a PIN, this feature must be **deactivated**. (Setting the PIN code to “0000” is not enough! The PIN code security feature must be fully switched off). Consult the manual for your mobile telephone for instructions on how to do this. If it is not possible to deactivate the SIM lock feature on your own phone, you will be unable to perform this step of the procedure. You must therefore use a handset that does not have*

a SIM lock, or visit a telecoms store to have the feature deactivated.

- Has the SIM card been activated?
 - *When you use a new SIM card for the first time, you must use it to make a call on your own mobile telephone. In order to do so, you must insert the SIM into your own telephone and then attempt to call another random number using your telephone. (see 6)*
- Has the SIM card expired?
 - *If you do not use or top up the SIM card for a period of six months or a year, it may be the case that the provider has blocked the SIM card. You will then need to purchase a new SIM card. For the precise terms and conditions, please refer to your SIM card provider.*
- Has the SIM card been inserted correctly into the holder?
 - *The SIM card only fits into the Birth Alarm Mobile Premium/Lite in one way. Check whether it has been inserted correctly. (see 5.3)*
- Can you make a call with the SIM card being used?
 - *Check whether you are able to connect to the GSM network with your own telephone and also check whether you can make a call. If the coverage offered by the provider is insufficient, it is possible that you may need to choose a different provider.*

The Birth Alarm Mobile Premium/Lite establishes a connection to the GSM network, yet does not make a call when tested.

- Have the numbers been programmed on the SIM card correctly?
 - *The numbers must be programmed in slots 11 and 12 of the SIM. If you have installed a new SIM card, you must switch the Birth Alarm Mobile Premium/Lite to programming mode and then (when the status LED is flashing blue-green) call the device. The Birth Alarm Mobile Premium/Lite will then automatically create the memory locations on the SIM card and save your number in the first of these.*
- Can your own telephone be reached?
 - *Try to call your own telephone using another.*
- Does the provider relate a top-up message prior to the call?
 - *When your credit is running low or has nearly expired, the call may be preceded by an automated spoken message from your*

provider, as a result of which the call is ultimately not transferred to your telephone (or is so only very briefly). Top up your credit using a top-up card or voucher, available from post offices and supermarkets. You can test this by inserting the SIM card into your own mobile telephone and calling a random number. You should then listen to see whether you hear a top-up message like this prior to the call.

- Is there enough credit?
 - *If the credit is very low or has been fully used, no further calls can be made. Top up your credit using a top-up card or voucher, available from post offices and supermarkets.*

I cannot deactivate the Birth Alarm Mobile Premium/Lite by returning the call.

- Is the voicemail for the Birth Alarm Mobile Premium/Lite's SIM card switched off?
 - *If the voicemail service for the SIM card is switched on, it is possible that this will be activated before the Birth Alarm Mobile Premium/Lite has been able to respond itself. You must switch off the voicemail service on the birth alarm's SIM card. Consult the manual or instructions supplied by the provider of your SIM card (you will have received these when you purchased your SIM card).*
- It is possible that the Birth Alarm Mobile Premium/Lite was simply busy calling you or another number on the SIM card.
 - *If you have waited a while before calling back, it is possible that the Birth Alarm Mobile Premium/Lite was busy making a new call. During this time, the Birth Alarm Mobile Premium/Lite is momentarily unable to accept incoming calls.*
- It is possible that you have rejected the call via the red button on your mobile and have called back immediately afterwards.
 - *Never reject an incoming call from the Birth Alarm Mobile Premium/Lite with the red button. Allow your telephone to ring out and do not answer! Only then may you call back.*

I have placed the Birth Alarm Mobile Premium/Lite on its side to test it, yet it does not begin calling.

- Have you switched on the sleep mode? (Note! Only the Birth Alarm Mobile Premium offers sleep mode)
 - *If you have activated sleep mode, the Birth Alarm Mobile Premium does not begin calling immediately after being turned*

on its side. In that instance, the Birth Alarm Mobile Premium simply responds to your mare becoming restless (e.g. contractions or briefly lying down). In order to switch off sleep mode, send an SMS stating "SL:0" to the Birth Alarm Mobile Premium.

- Has the Birth Alarm Mobile Premium/Lite recently issued a notification?
 - *If you are testing the Birth Alarm Mobile Premium/Lite and have called it back in order to deactivate it, it must then be returned to lying in a flat position for approximately one minute before it will start issuing notifications again. This prevents the Birth Alarm Mobile Premium/Lite from continuing to call after having just been deactivated.*

It took a few minutes before I was called the first time.

- Have you just changed numbers on the SIM card, or have you installed a new SIM?
 - *If you have programmed the SIM card manually and you have not done so correctly, the Birth Alarm Mobile Premium/Lite will itself attempt to store the numbers you programmed on the SIM card correctly. The call will be transferred to your telephone more quickly the next time.*
- Have you installed a new SIM card and not removed the PIN code?
 - *If you have not deactivated the PIN code request feature of the SIM card, the Birth Alarm Mobile Premium/Lite will itself attempt to deactivate the PIN. This will only be successful, however, if the PIN code is "0000". (see 6)*

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